ALGINATE REPLACEMENT

Addition Curing Silicone Impression Material

Alginate Replacement is a medium viscosity vinyl polysiloxane elastomeric impression material for use as an alginate alternative.

INDICATIONS FOR USE

- Impressions for the production of temporary crowns and bridges
- Orthodontic impressions
- Surgery impressions
- Impressions for models for construction of trays for whitening solutions
- Impressions for modifying / expanding removable prosthesis
- Impression for opposing models

TECHNICAL DATA	ALGINATE REPLACEMENT
ISO 4823	Type 2, medium-bodied consistency
Mixing Ratio (base : catalyst)	1:1
Mixing Time	Auto-Mix System
*Total Working Time (minutes)	1:00
Time in Mouth (minutes)	2:00
*Total Setting Time (minutes)	3:00
Strain in compression % (min-max)	3-5
Elastic recovery (%)	>99.5
Linear dimensional change (%)	<0.2
Shore A hardness	45±2

***ATTENTION!** Application is at 23°C ± 2°C / 73°F ± 4°F, 50 ± 5% relative humidity Higher temperature reduces working and setting times and lower temperature prolongs them.

Intended Use: Dental Impression material

Use environment: Dental practice

Patient target group: People treated in the course of a dental procedure with no limitations in age or gender.

IMPRESSION TRAYS: All types of trays whether perforated, non-perforated or special can be used provided that they are non-deformable and rigid. In order to avoid errors during impression taking, especially during removal from the oral cavity, it is very important to ensure proper adhesion of the material to the impression tray. If using an adhesive, ensure the type of adhesive selected is appropriate to the tray used (refer to the corresponding instructions of use of the adhesive).

APPLICATION: 50mL Cartridge (1:1)

Load the cartridge onto the appropriate hand-held dispenser following the manufacturer's instructions. Once loaded remove cap and extrude material until it exits both chambers at the same rate and then mount the appropriate mixing tip. An intraoral tip may be placed on the mixing tip. To avoid air bubbles always keep tip of the mixing tip immersed in the impression material. Fill the prepared tray with Alginate Replacement. Insert tray into mouth and leave the material to set for the recommended setting time. After use either leave the used mixing tip as a cap until the following application or carefully remove mixing tip and place cap immediately taking care not to cross contaminate the materials. Intraoral tips and mixing tips are for single use only. After use either leave the used mixing tip as a cap until the following application or carefully remove mixing tip and place cap immediately taking care not to cross contaminate the materials. Mixing tips are for single use only.

DISINFECTING: Once impression is removed from mouth, rinse under lukewarm water. The impression can then be disinfected with appropriate disinfecting solutions or sprays such as 2% glutaraldehyde. Take care to follow the manufacturer's instructions for use. For further information regarding disinfecting impressions: JADA 1991; vol. 122; IS(8); p110. If disinfecting with hydrogen peroxide, thoroughly rinse with lukewarm water to avoid the formation of bubbles.

CASTING: The dry and clean impression can be cast between 30 minutes and two weeks after removal. All type III dental plasters (e.g. Moldano, Heraeus Kulzer) and IV dental plasters (e.g. Fujirock EP, GC) and standard model-casting acrylics can be used.

ELECTROPLATING: The impression can be electroplated using copper or silver baths

PRECAUTIONS: • Contact with latex gloves may impair the setting reaction. Vinyl gloves are recommended • Eugenol or certain haemostatic materials may impair the setting reaction • Do not combine with condensation curing silicones, polyether or polysulfide • Avoid spillage of material on clothing as it cannot be removed chemically • Avoid contact of material with eyes. In case of contact rinse eyes with plenty of water and consult a physician immediately • Seek immediate medical attention in case of ingestion • Do not use after the use-by date

CONTRAINDICATIONS: Application of the above material is contraindicated:

- If a patient is known to be hypersensitive or allergic to any of the ingredients

STORAGE: Store material between 10 - 25°C (50 - 77°F). Store impression at room temperature (max. 25°C/77°F).

LOT NUMBER AND EXPIRY DATE: The above are printed on each package and should be quoted in all correspondence which requires identification of the product.

DISPOSAL: Dispose of the product according to local regulations as applicable

USERS / FOR PROFESSIONAL USE ONLY: These materials have been developed for use exclusively by dental professionals and must be handled according to the instructions of use. The recommendations included in the instructions correspond to the actual state of knowledge regarding dental techniques. This does not relieve the user from determining the suitability of the product for its intended purpose. The manufacturer cannot be held responsible for damages caused by other uses or incorrect handling. Caution: Federal USA law restricts this device to sale by or on the order of a dental professional.

REPORTING OBLIGATIONS: Should a serious incident in relation to this product occur, it should be reported to the manufacturer and the competent authority.

YOUNG
INNOVATIONS


Distributed by: Young Innovations, Inc.
2260 Wendt Street Algonquin, IL 60102 USA
www.younginnovations.com • (800) 325-1881
Made in Greece

L5-VP-9200 Rev. G Rev. Date 07/30/2025

Rx
ONLY

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A DENTAL PROFESSIONAL.

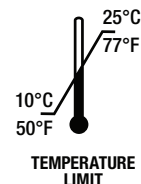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Distribué par: Young Innovations, Inc.