EN 2Tone™ Disclosing Solution

Intended Use: To enhance visualization of the presence of plaque on tooth surfaces.

Indications for Use: For general population of dental patients (Children, at least 5 years old with adult supervision).

Contraindications: Not intended to diagnose or treat any condition. For use as an educational aid only. For professional use only.

Report any serious incident related to device use to Young Dental and the respective Member State competent authority. Dispose according to dental facility and municipal guidelines.

2Tone is a trademark of Young Dental.

Directions:

1. Paint on teeth with cotton applicator or place 3-4 drops under tongue and wipe teeth with tongue.
2. Rinse mouth to remove excess solution.
3. Stains old plaque blue. Stains new plaque red.
4. Brush teeth and floss thoroughly to remove all stain.
5. Replace cap tightly after use.

 Keep out of the reach of children. Do not swallow.

 Do not use beyond Expiration Date printed on label.

Contains: FD & C Blue #1, D & C Red #28

 Manufactured by:
Young Dental Manufacturing Co.
13705 Shoreline Court East
Earth City, MO 63045 USA
800-325-1881
www.younginnovations.com

Young Microbrush Ireland, Ltd.
Clogherane, Dungarvan
Co. Waterford, X35 VE02, Ireland

UK Responsible Person
Astek Innovations Ltd
95 Atlantic Street, Altrincham
WA14 5DH United Kingdom
Tel: +44 (0)161 942 3900

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau, Switzerland

Made in the USA 

Symbol	Definition	Symbol	Definition	Symbol	Definition
	Consult instructions for use		Medical Device		Importer
	Manufacturer		Use by		Authorized representative for Switzerland
	UK Conformity Assessment mark		Caution: Federal Law restricts this device to sale by or on the order of a dentist		Authorized representative in the European Community
	Non Sterile		Caution		CE marking Conformity
	Batch or Lot code		Product reorder number		Unique Device Identification
	Made in the USA				

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